

# Veterinary Fluid Warmer Vet Temp 3



**Vet Temp 3 fluid warmer supplies external heat to I.V. set fluids, near to the patient's body temperature.**

- Use with standard fluids, blood and TPN fluids
- Light weight, 200 gr only!
- Accepts standard tubing sets for small animals
- Fast and easy to set-up
- Easy to clean
- Over-temperature protection, five alarms including high and low temperature
- Two attaching methods: on I.V. stand lanyard and bracket



# Vet Temp 3 Fluid Warmer

The front and back of the device are joined by 2 hinges.

Simple construction and shock resistant casing for a high durability.

The operator can adjust the setting temperature in the range of 35°C - 42°C according to actual needs.



## Why investing in a veterinary fluid warmer?

Maintaining the body temperature is very important during and after an anaesthesia for avoiding serious consequences such as:

- Risk of shock
- Risk of heart problems, renal insufficiency in a long term ...
- Risk of hypothermia


Vet Temp 3 helps to decrease the temperature difference between the animal and the I.V. liquid. The animal needs less energy for maintaining his body temperature.

Similar to the Carbon Tech heating mats, the Vet Temp 3 Fluid Warmer avoids the patient's temperature cools down too much.


Fields of applications and benefits:

- ▶ Safety infusion of a warm fluid
- ▶ Suitable for infusions with a pump or for traditional infusions by gravity
- ▶ Easy to use
- ▶ Cost effective and durable device
- ▶ No need of disposables


Three lengths of heating groove design to make infusion heating more flexible and accurate:

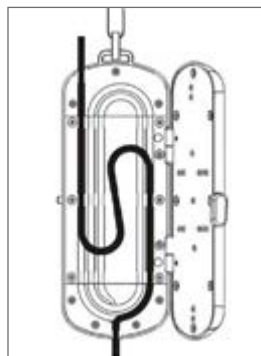

25 cm groove

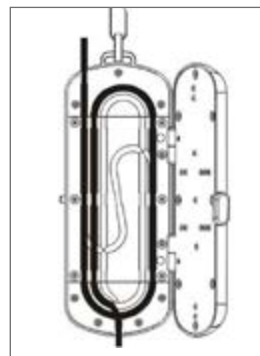

31 cm groove

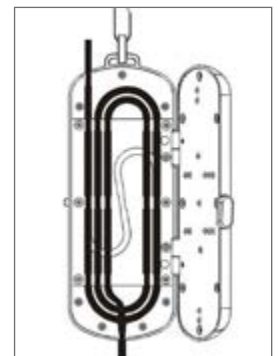

52 cm groove

Technical data:

▶ Heating groove lengths	25, 31 and 52 cm
▶ Applicable I.V. set	Exclusive I.V. set or standard PVC I.V. set (20 and 60 drops / ml)
▶ I.V. set outer diameter	Between 3.4 mm and 4.5 mm
▶ Temperature	Between 35°C and 42°C
▶ Warming up time	≤ 2 min.
▶ Alarms	Overheat & low temperatures, system error, door open, heating reminder
▶ Power	100-240 V / 50-60 Hz
▶ Dimensions and weight	18x7x3.5 cm - 200 gr



Mano  
Médical  
by génia

22 rue Charles de Gaulle  
22100 Lanvallay - France  
Tel. +33 (0)2 96 85 86 76

export@manomedical.com

www.manomedical.com